



Pregnant? Confirm it with milk.

Optimize reproductive efficiency with the IDEXX Milk Pregnancy Test.

Now you can offer your customers even greater value using existing cow and goat milk samples. The IDEXX Milk Pregnancy Test is the simplest, most cost-effective way to confirm open cows and goats to manage fertility and overall herd health management programs.

Accurate determination of pregnancy status. The IDEXX Milk Pregnancy Test delivers a high level of sensitivity and specificity from ≥ 28 days postbreeding and day 60 postcalving (cows).

Trusted IDEXX ELISA technology. Seamlessly integrates into existing lab-based work flows, allowing you to offer greater value to your customers at little impact to your operation.

Timely results. Uses proven IDEXX ELISA technology for trusted results in less than 3.5 hours.

Cost-effective and hassle-free way to maximize overall herd health management programs.

.....
Learn how to maximize the value
of milk samples and optimize your
business today.
.....

The IDEXX Milk Pregnancy Test

Give your clients greater value with accurate, easy confirmation

- Boost client confidence—and your reputation—with definitive confirmation of pregnancy status in cows and goats.
- Demonstrate that you're a trusted partner in reproductive health by helping clients improve reproductive performance and shorten calving intervals.
- Run more pregnancy tests because clients trust your results, value the quick turnaround, and rely on you for confirmatory testing.
- Expand your testing options, offering customers a comprehensive herd health management program using existing samples.

Performance on par with ultrasound/palpation

		Ultrasound/Palpation	
		Pregnant (n=1,182)	Open (n=658)
IDEXX Milk Pregnancy Test	Pregnant	1,122	36
	Recheck	45	20
	Open	15	602

Sensitivity: 98.7%*
Specificity: 94.4%*
Rechecks: 3% of total tested

*Performance of the IDEXX Milk Pregnancy Test was determined by comparing test results to those from ultrasound/palpation (N=1315 cows.) Rechecks were not included in performance calculations. See IDEXX Milk Pregnancy Test validation report for full test performance data.¹

		Ultrasound/Palpation	
		Pregnant (n=63)	Open (n=92)
IDEXX Milk Pregnancy Test	Pregnant	56	0
	Recheck	4	0
	Open	3	92

Sensitivity: 94.9%*
Specificity: 100%*
Rechecks: 2.6% of total tested

*Performance of the IDEXX Milk Pregnancy Test was determined by comparing test results to those from ultrasound/palpation (N=160 goats.) Rechecks were not included in performance calculations. See IDEXX Milk Pregnancy Test validation report for full test performance data.¹

For more information about the **IDEXX Milk Pregnancy Test**, please contact your IDEXX representative or visit idexx.com/reproduction.

IDEXX Laboratories, Inc.

One IDEXX Drive
Westbrook, Maine 04092
USA

Tel: +1 207 556 4890 or
+1 800 548 9997
Fax: +1 207 556 4826 or
+1 800 328 5461

IDEXX Europe B.V.

Scorpius 60 Building F
2132 LR Hoofddorp
The Netherlands

Tel: +31 23 558 70 00 or
+800 727 43399
Fax: +31 23 558 72 33

IDEXX Laboratories, Inc.

3F-5 No. 88, Rei Hu Street
Nei Hu District
11494 Taipei
Taiwan

Tel: +886 2 6603 9728
Fax: +886 2 2658 8242

Test with Confidence™

1. Data on file at IDEXX Laboratories, Inc. Westbrook, ME USA.